Instructions for Use

Titanium Hemostatic Clip

ENGLISH

IMPORTANT!

This booklet is designed to assist in using this product. It is not a reference for surgical techniques.

Description and Indications for Use

The titanium hemostatic clip is supplied packaged EO sterile and has been specifically designed to securely grip the vessel or tissue to provide complete occlusion a traumatically. User shall select appropriate clip size according to procedure ensuring that all tissues are completely encompassed by clip.

Contents

P/N	DESCRIPTION	COLOR CODE
350-000-101 Use with Small Applier	20 Cartridges of 6 Small Clips	Blue
350-000-102 Use with Small Applier	20 Cartridges of 6 Medium Clips	White
350-000-103 Use with Medium/Large Appliers	20 Cartridges of 6 Medium/Large Clips	Green
350-000-104 Use with Large Appliers	20 Cartridges of 6 Extra Large Clips	Yellow

Contraindications, Warnings, and Precautions

This product is not intended for use whenever endoscopic techniques are contraindicated.

Attention – Do not use if package is damaged or open.

Attention – Product is for single use only – DO NOT RE-STERILIZE.

Important – This booklet is designed to provide instructions for the use of hemostatic clips. It is not a reference to surgical endoscopic techniques. Before using product, read the following information thoroughly. Failure to properly follow the instructions may lead to serious surgical consequences.

Important:

- This clip is not intended for Fallopian Tube Ligation.
- As a non-magnetic marking clip, it has low magnetic susceptibility and is safe for use in the MRI environment. Positional errors and artifacts may increase dramatically with increasing static magnetic field strength and/or increasing field of view size. Artifact size may vary greatly for different MRI pulse sequences. The titanium clip is biologically inert and can remain in vivo without sequela.

Warnings:

To prevent premature closure or release of the clip:

- Never grasp the applier by both shanks when loading the clip.
- Do not squeeze applier handles before jaws are placed around vessel or tissue.
- Ensure appliers are fully closed to handle stop point when applying clip.
- Inspect ligation site to ensure secure application and hemostasis.

To prevent damage to applier jaws:

- Avoid introducing applier tips at an angle into the cartridge rack, this may misalign tips.
- Inspect applier and jaws before each use.
- Use recommended appliers.
- Open applier for thorough cleaning and sterilization.
- Federal (USA) law restricts this device to sale by or on the order of a physician.

Instructions

1. Inspect the integrity of the individual sterility protection (packaging) before using.
2. Open the sterile packaging.
3. Remove the protective strip from the adhesive base of the clip cartridge and press firmly on a stable surface.
4. Select applier according to clip size (color coding).
5. Hold applier at the box lock or grasp one shank of the applier in the palm with forefinger at the box lock.
6. Introduce the jaws of the applier vertically into the cartridge clip rack.
7. Push the applier all the way into the cartridge rack and withdraw to remove the clip.
8. Place the loaded clip and jaws of the applier around the vessel or tissue to be occluded.

9. Firmly squeeze the handles together closing the clip. The clip closes tip to tip preventing vessel slippage before fully closing.
10. Release the handle and remove applier.

Storage

Store the unopened sterile package in a cool dry area, with good ventilation away from areas of extreme temperature and humidity

Shelf Life

Contents are sterile if package is unopened, undamaged, and within the validity date printed on the package. THIS CLIP IS INTENDED FOR SINGLE USE AND CANNOT BE RESTERILIZED. Do not use if package is opened or damaged. The manufacturer declines responsibility for problems resulting from improper use. Prices, specifications, and model availability are subject to change without notice.

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**REMAINDER OF PAGE
INTENTIONALLY LEFT BLANK**

This product may be covered by one or more U.S. patents and their foreign counter parts. Other Patents may be pending.

GENICON and the GENICON logo design are U.S. registered trademarks of GENICON Rights Reserved.

Disclaimer of Warranties on Re-Processing or Re-Sterilizing of Vendor's Disposable (Single-Patient) Products: Any reprocessing or re-sterilizing of disposable single patient devices voids Vendor's warranty.

Vendor assumes no liability for reprocessed or re-sterilized products. Vendor warrants that reasonable care has been used in the manufacture of these disposable single patient devices and that they are free from defects in workmanship or materials at the time of shipment from Vendor. Vendor's sole obligation shall therefore be to repair or replace any device which it determines was defective at the time of shipment. Because no product is completely effective under all circumstances, and because the actual use and handling of these devices are beyond Vendor's control, Vendor cannot warrant for a good effect or against a bad effect in the application and use of these disposable single patient devices. The Purchaser therefore assumes all liability resulting from re-sterilization or re-processing of these disposable single patient products. Vendor therefore gives no warranty of merchantability or fitness for a particular purpose. Vendor will not be liable for any direct, indirect, incidental or consequential damages resulting from or related to a disposable product that has been re-sterilized or re-processed. Vendor shall not be liable for incidental or consequential loss, damage or expense resulting from the use or application of these said products. This warranty is in lieu of all other warranties, whether implied, express, oral or written, and no individual or entity has the authority to vary the terms of this warranty.

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